


EYEZEN™+ 0 CUTOUT CHART

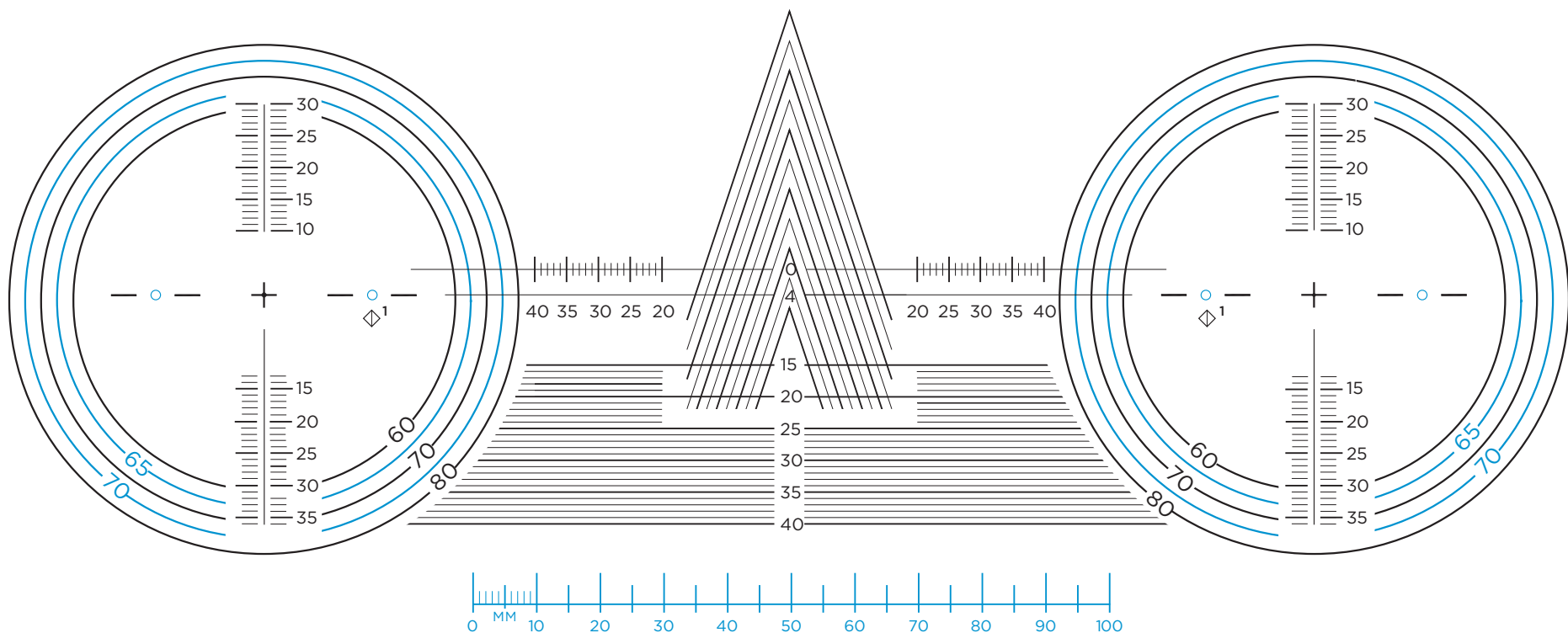

Please verify correct size with a ruler

◇¹ Lens Engraving symbol location

PRINTING INSTRUCTIONS

To print this measuring chart at the correct size from Acrobat Reader:

1. Go to **File**, then select **Print**
2. Under **Page Sizing & Handling** options, select **Actual Size**
3. Click on **Print** button to print

Note: Chart measurements will be incorrect if "Fit" or other page-scaling options are selected. It is recommended to verify the printed chart is the correct size with a ruler.

LENS ENGRAVING	
PRODUCT	SYMBOL
Eyezen+ 0 (0.00 D)	◇

BLOCKING AND FINISHING: EYEZEN™+ 0 LENSES

Eyezen™+ enhanced single vision lenses are an everyday single vision lens.

Check with your supplier to ensure that your finish blocker software contains the necessary product files for Eyezen+ lenses, so the layout calculations place the fitting cross correctly at the specified distance PDs and fitting height.

VERIFICATION

Instructions to verify power in Eyezen+ 0 lenses

- 1 Verify the prescribed power at the fitting cross
- 2 If prism was prescribed, then verify prism power at the fitting cross
- 3 Confirm engraving to ensure Eyezen+ 0 was provided



Including a fitting height is recommended, as Eyezen+ 0 design is a digital aspheric single vision lens.

Go to [EyezenPro.com](https://www.eyezenpro.com) to find more resources and information on Eyezen+ lenses.

Eyezen™+